

PANSEMENT SPRAY

LE PRODUIT :

- Ce produit s'applique sur les blessures superficielles, il protège la peau des germes et des bactéries et soulage la douleur par son rôle protecteur mécanique. En quelques instants, il génère un film telle une seconde peau sur la zone à traiter. Le film est résistant à l'eau.

APPLICATIONS :

- Egratignures, éraflures et irritations superficielles.
- Coupures bénignes.

ACTIVITES :

- Favorise la cicatrisation.
- Réduit le risque d'infection.
- Protège et Adhère à la peau.
- Autolubrifiant ,
- Résiste à l'eau.

MODE D'EMPLOI :

- Nettoyer et sécher soigneusement la zone à traiter.
- Bien agiter avant l'utilisation.
- Pulvériser uniformément à une distance de 20 cm par jets successifs et brefs.
- Vérifier que la zone est parfaitement recouverte.
- Laisser sécher.
- La protection formée se retire à l'alcool ou se désagrège naturellement, le reliquat peut-être retiré à l'eau savonneuse.



PRESENTATION :

- En aérosol de 100 ml ref 05015

COMPOSITION :

Dimethylether, Methylmethacrylate/Butylmethacrylate Copolymer, Methylal, Polyvinyl, Caprolactam, Calamine, Disteardimonium Hectorite, Allantoin, Triclosan, Cyclopentasiloxane, Dimethyl Adipate, Glutarate and Dimethyl Succinate, Benzyl Alcohol, Fragrance, Geraniol, Hexyl Cinnamal, Limonene, Linalol, Alpha-Isomethyl Ionone, Buthylphenyl Methylpropional.

S.A. Eurodist N.V.

143 chaussée de Namur 1402 Thines (Belgium)
 Tel : 32 67 63 49 90 Fax : 32 67 84 11 84
 www.eurodist.be E Mail : info@eurodist.be